

# Extra Wide Bariatric Shower Seat Installation Instructions

## CONSTRUCTION

Stainless Steel, 28mm OD White Frame with Folding Legs and Adjustable Feet.

**WALL FIXINGS ARE NOT SUPPLIED.**

## IMPORTANT

The shower seat brackets can only be secured to a sound brick or concrete wall. If the wall is constructed from any other material, please seek further advice – the manufacturers cannot be held responsible for any deviation from this.

## DIRECTIONS FOR INSTALLING EXTRA WIDE SHOWER SEAT:

1. Position the shower seat brackets against the wall at the height of 450mm from the floor to the underside of the seat.
2. Mark the position for 2 holes on each bracket (making sure both brackets are parallel at all times)
3. When the wall has been drilled and plugged according to the structural requirements\* place the shower seat against the wall and locate fixings\* into holes – making sure the 2 wall brackets are parallel and are screwed squarely against the wall.
4. Make sure fixings are secured firmly into wall
5. Adjust and level the feet as required.

## USING THE EXTRA WIDE SHOWER SEAT:

During use, it is important that both legs and feet are pulled forward so the legs are in the vertical position and placed securely flat on the floor. Failure to do so may damage the seat fixings. Never use with the legs uneven or slanting, this will compromise the safety of the user.

**\*Wall fixings are not supplied with this product due to the variety of structural conditions existing. The installer should use fixings appropriate to the mounting structure.**



## **IMPORTANT**

**MAXIMUM SEAT LOAD WHEN FITTED CORRECTLY:**

**50 Stone**



# Extra Wide Bariatric Shower Seat

## Care & Maintenance

1. Please examine your shower seat for any visible damage before using. If you see any damage or suspect a fault, please contact your supplier.

**In this case, please do not use your product as it may compromise your safety.**

2. **Cleaning:** All of the product components are manufactured from quality materials. However, it is very important that you only clean your shower seat using a non-abrasive cleaner or mild detergent with a soft cloth and avoid using lime scale remover. Abrasive cleaners and or abrasive cleaning pads will seriously damage your product beyond repair. **Do not exceed 50°C when cleaning the seat as this will permanently damage the plastic.**

**NOTE:** Regular daily cleaning will reduce the build-up of lime-scale on your product.

Under **No Circumstances** use any abrasive cleaners or materials on any of the components. Also check that the cleaner you intend to use is suitable for ABS, Vinyl, Polymer Plastics and Thermo plastic rubbers and Polyurethane Rubber.

### 3. Use of Bath Oils

We do not recommend the use of bath oils but if used, caution should be observed if you are using any form of bath oils or oily skin preparation with your shower chair as this may impair your natural contact skin resistance.

In the event of using any such oil, your seat must be thoroughly cleaned with a suitable detergent to break down any oily residue. Failure to observe this recommendation could impair your safety and the function of the shower seat.

